


AWEST



AirWash

ChairSide Cleaning Machine

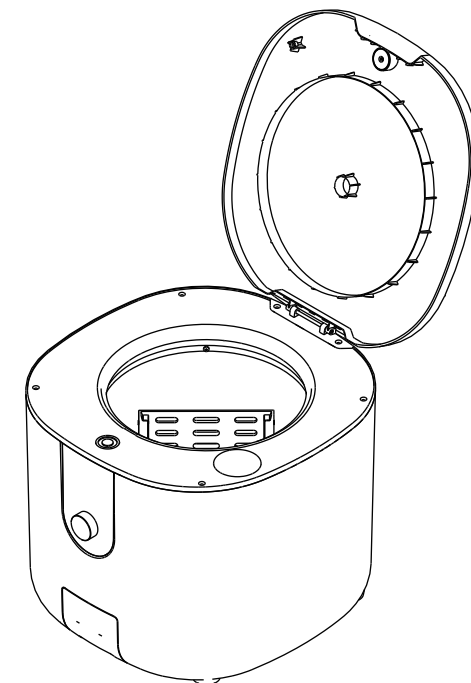

🌐 [heygears.com](https://www.heygears.com) ✉ info@heygears.com
☎ Tel: +1 (318) 353-4295 (Global)/ +1 (949) 418-9418 (USA) / +44 148-396-8549 (EU)
📍 USA-17931 Sky Park Circle, Suite E, Irvine, CA, 92614
CHN-Block B2, 501, 601, Enterprise Accelerator, Kaifa District, Guangzhou City

Follow us @HeyGears



Please read this installation and operation manual carefully before use

Thank you for choosing HeyGears. **For your safety, please follow the operating instructions.** If you fail to comply with the instructions in this manual, HeyGears will not be responsible for the consequential losses unless required by law.

Please keep this operation manual properly for future reference. The pictures are for reference only and the products are subject to the actual product.

1 Items & Parameters



AirWash



Power Code



Waste Cabinet



Brush

Name	UltraCraft Washing Unit	Input	100–240 V~ 50/60 Hz
Model	UltraCraft AirWash	USB Output	5 V == 500 mA
Size	388 x 403 x 349 mm	Power	154 W
Weight	15.5 kg		

2 Overview



3 Installation Requirements



Place the product horizontally on a stable table before use.



Make sure there is a 100V–240V power outlet nearby.



Open flame is not allowed in the environment where alcohol is used.



Do not put your hand into the cleaning cabinet during the operation.



If you observe any obvious stains in the waste cabinet, you need to use a brush to clean it.



If the display shows that the waste cabinet is full, you need to take it out to clean, and you should not take it out too fast to prevent the waste resin from spilling.

4 Operation Steps


STEP 1

Plug the supplied power cord into the printer power port and turn on the switch next to it.



STEP 2

Open the lid and insert the build platform with printed parts into AirWash along the track, making sure it is inserted in place. Pour about 1L of 95% concentration alcohol into the resin filling port for the first time until the liquid level just diffuses through the filling hole, then close the filling port protection lid.

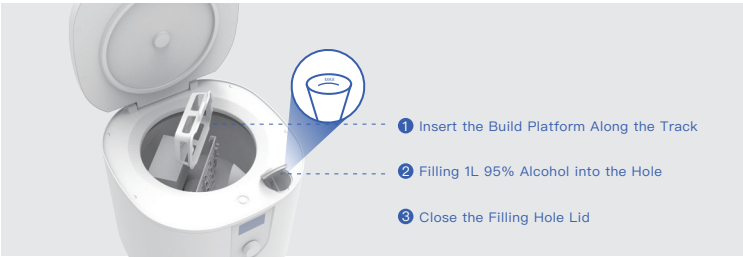

STEP 3

Close the lid, and then rotate the knob to select the model material according to the tips on the screen, and then press it to start washing.
*You can long press the knob for canceling.



STEP 4

After washing, you can open the lid when the knob light flashes blue, then take out the build platform and move to the post-curing steps.



5 Maintenance and Technical Support

You can contact the after-sales and technical support staff via the following ways:

1. After-sales support email address: support@heygears.com
2. Available worldwide: +1 318–353–4295
3. Europe: +44 148–396–8549
4. USA: +1 949–418–9418
5. Visit <https://www.heygears.com/services>